## Comprehensive Therapy for Neuropathy Pain—Feet and Hands

**With Optional Circulation Support Pad Placments** 

Boot 122 pads used alone will address most neuropathy pain. However, many patients can benefit from additional supportive pad placements

## **Boot 122 Pads**

- Use strapped down the back of the calves and across the bottom of the feet
- Also use folded around the foot and ankle like a "boot" for more concentration on the foot

## **Local 132 Pad** or alternately the Boot/122

- ◆ Lumbar area over L4/5
- Groin on both sides at the femoral arteries and over lymph glands, lymph and blood circulation
- **♦ Thighs** for circulation

## **For Neuropathy in the Hands**

- For the hand and forearms –rest the hand and wrist on the Local 132 pad for 10 minutes
- Use the Local 132 pad on the cervical and upper thoracic area for 20 minutes. Also support circulation in lymph areas if edema is present: over the collarbone and under arm areas.
- **HINT: Strap the strap to itself, not the pad.** Connect the gripper end of the strap to the loopy side of the strap instead of attaching directly to the pad forming a loop—it is easier to remove.

